PREPARATION

Wash the tub where the Bath Safety Rail will be installed. All contact surfaces must be clean, dry and free of residue to ensure a secure attachment.

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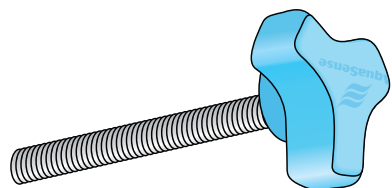
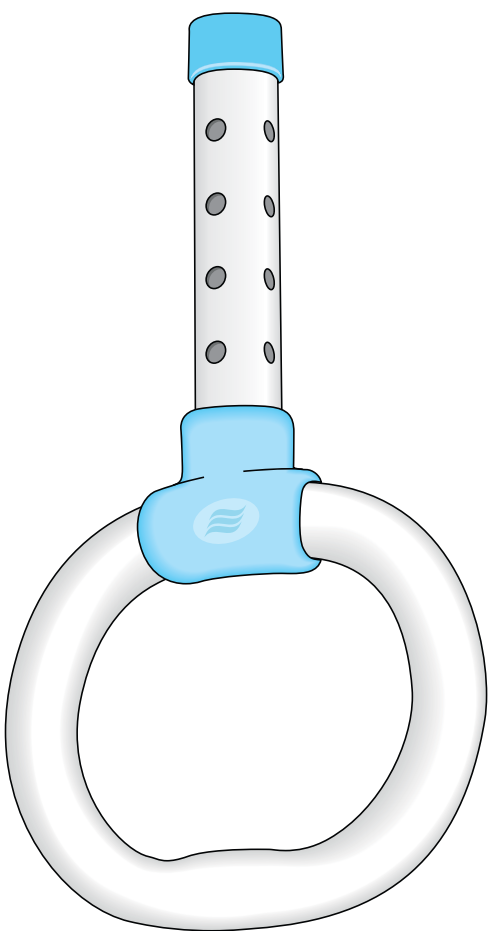
EN

- Before setup and use, verify that all parts illustrated are present and examine them for any damages that may have occurred during shipping. Do not assemble this product if there is apparent damage.

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EN

This AquaSense® product is guaranteed to be free from defects in materials and workmanship for the life of the original user. This warranty does not extend to non-durable components which are subject to normal wear and replacement. Any alterations, abuse, misuse, or accidental damage voids this warranty. For replacement or repair under warranty, please call:

CANADA & INTL: 866-752-1363

USA: 866-694-5085

For more information on this, or any other AquaSense® product, visit: www.myaquasense.com

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Tool-free setup • Assemblage sans outils • Montaje sin herramientas

A ASSEMBLE THE GRIPPING CLAMP

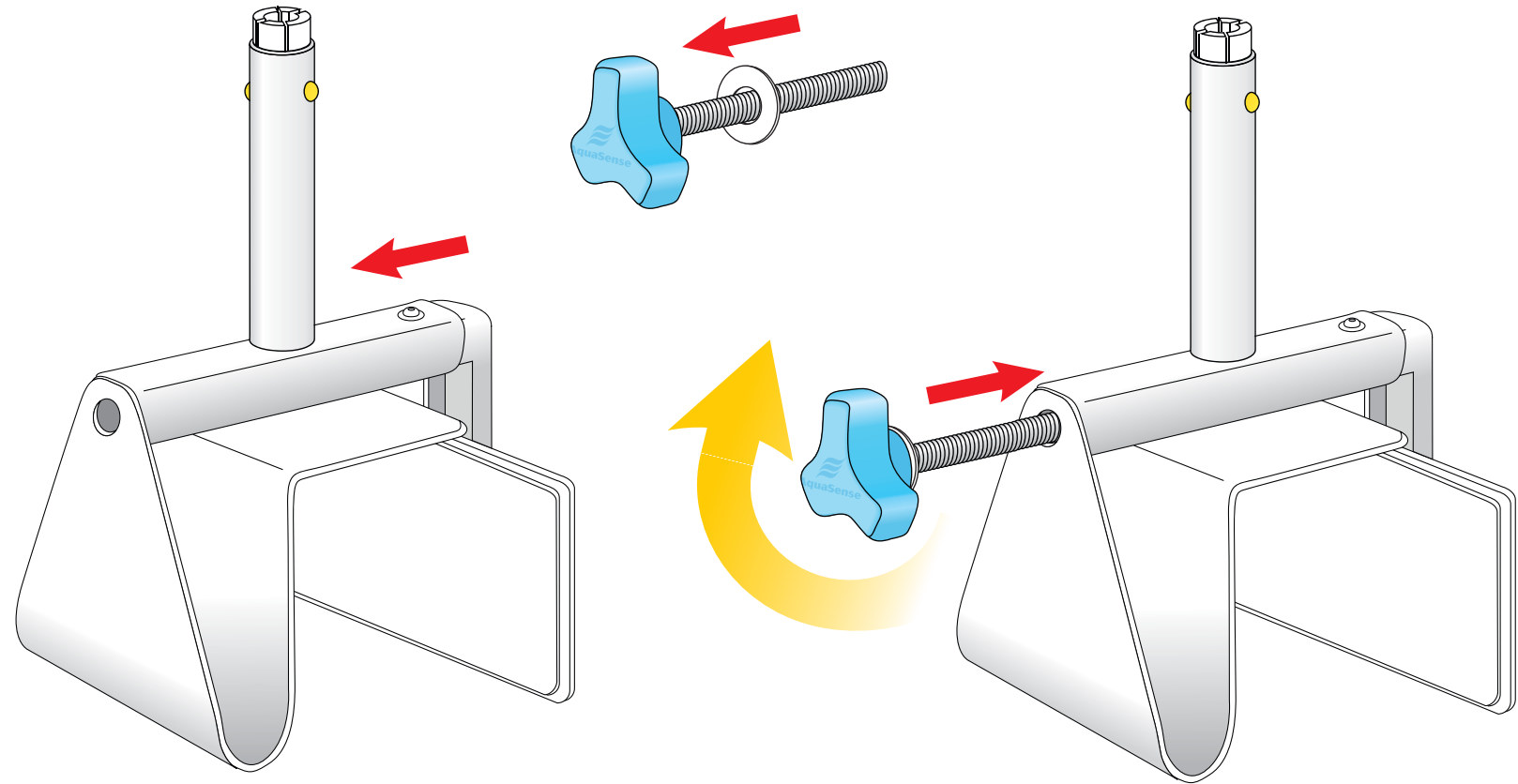
- Push the Mobile Gripping Clamp plate inward until the screw grooves are visible.
- Place the white plastic washer on the adjustment knob screw, near the head of the screw.
- Insert the screw in the Gripping Clamp hole and rotate until it has been properly gripped by the threads.

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B LOCKING THE GRIPPING CLAMP ONTO THE TUB

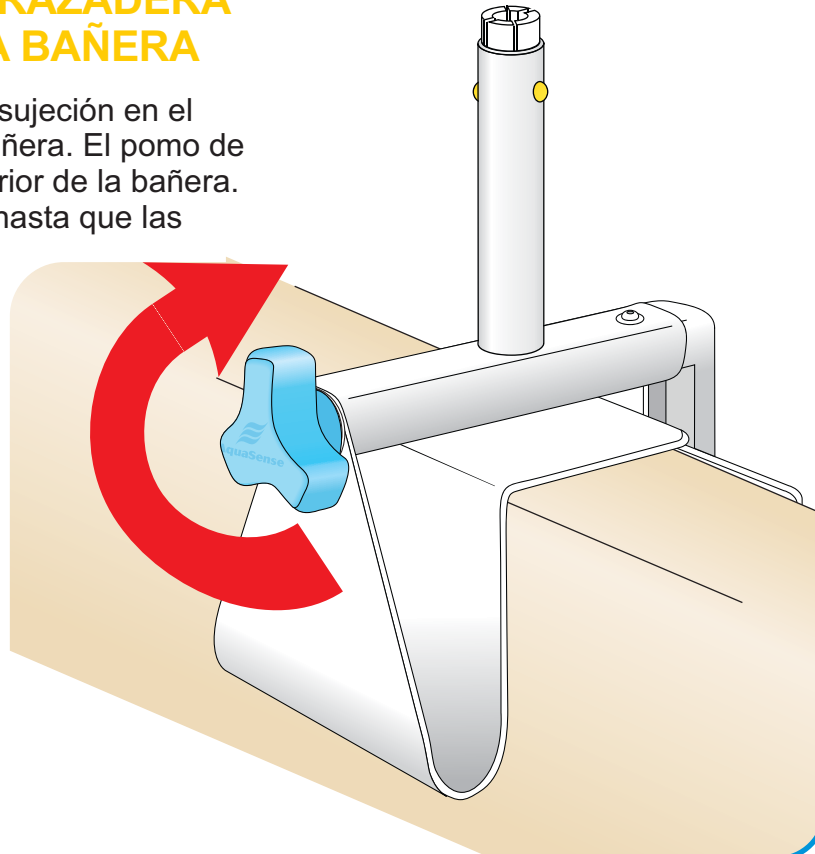
Place the Gripping Clamp on the bath tub wall, at the desired location. The adjustment knob must be positioned outside the tub. Tighten the Adjustment Knob until the fixed and mobile parts of the clamp become snug. Carefully continue to tighten the clamp until the bath rail is securely attached to the tub wall. **CAUTION: Do not over-tighten as this may damage your tub.**

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C ADD THE HANDSET EXTENSION

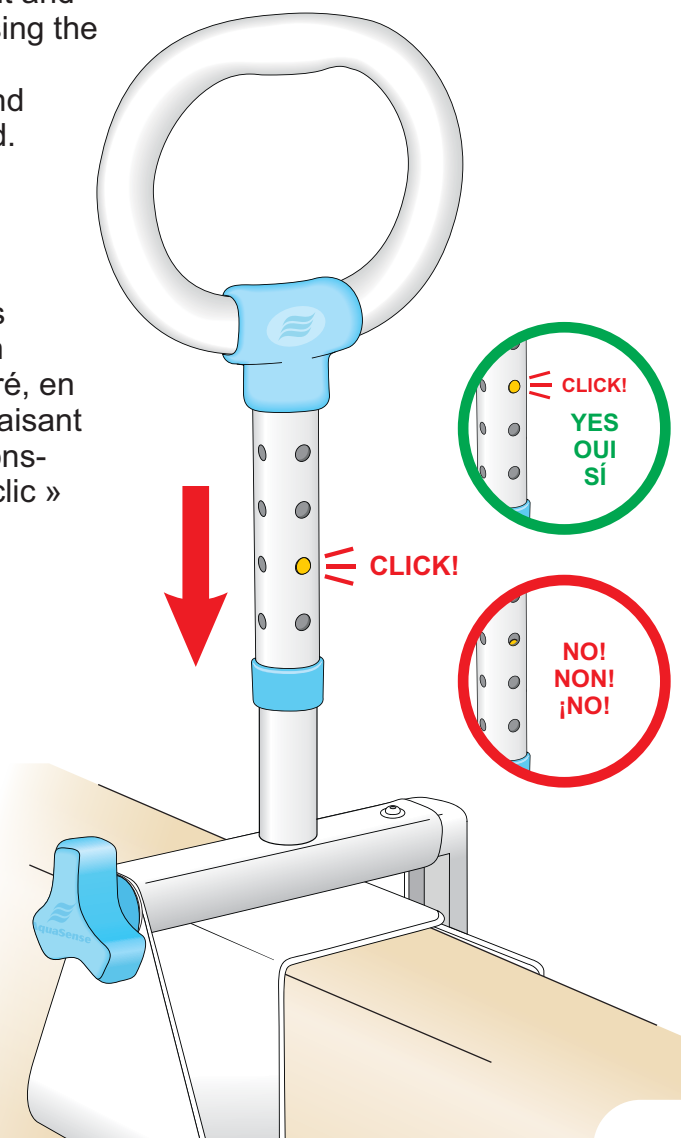
Place the Multi-Orientation Height Adjustable Handgrip onto the Metal Clamp Shaft, as illustrated. Lower the Handgrip until the push buttons lock it into place. Adjust the Handgrip at the desired height and orientation (parallel or perpendicular to the bath) by pressing the push buttons simultaneously and sliding it on the primary metal shaft. The push buttons will automatically deploy and lock the Handgrip into place when a "click" sound is heard.

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D TEST THE MOUNTING / STABILITY OF THE BATH TUB RAIL

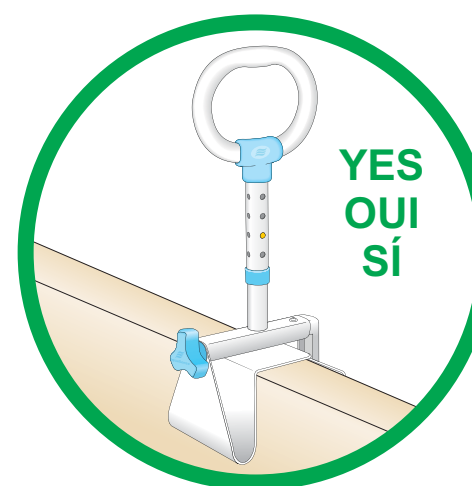
Test the mounting of the product by pulling on it from side to side and from front to back. If it moves, tighten again in the desired position.

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Precautions

EN

- Be careful to slowly tighten the adjustment knob to gradually engage the Gripping Clamp. Do not over-tighten on glass and/or porcelain tubs as it may crack or damage them.
- Verify that the Bath Safety Rail is properly secured by pulling it from side to side and from front to back. If it moves, tighten again in the desired position. **Your safety rail may require tightening from time to time. Check periodically and tighten again when needed.**
- Use a mild household cleaner to maintain your product. Avoid the use of harsh solvents, abrasive powders and scouring pads.

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