


drive™

www.drivemedical.com

deluxe bariatric shower chair with back

item # 12021KD-1

BEFORE YOU BEGIN

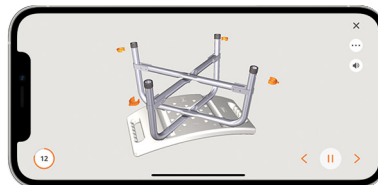

Scan for Easy 3D Instructions

Download the BILT app before you begin



EU Authorized Representative
Y Sung Handelsvertretung
Duesselthaler Str. 24 40211
Duesseldorf, Germany





www.drivemedical.com

drive™


parts diagram/list

Weight capacity 500 lbs (226 Kg)

A phillips head screw driver and a wrench are required to assemble this unit.

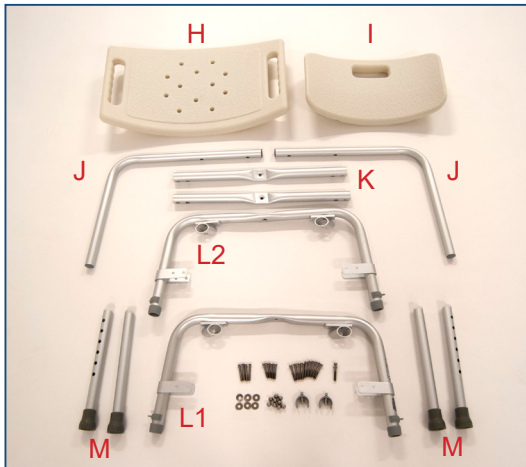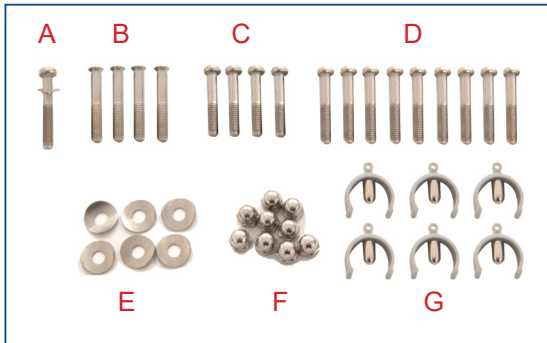


- | | |
|---------------------------|-------------------|
| A- Long Center Screw (1) | H- Seat |
| B- Flat Head Screw (4) | I- Backrest |
| C- Backrest Screws (4) | J- Backrest Frame |
| D- Cross Brace Screws (9) | K- Cross Brace |
| E- Washers (6) | L- Leg Receivers |
| F- Nuts (9) | M- Legs |
| G- U-Clips (6) | |

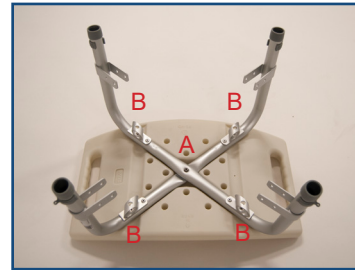

1. Assemble the leg receivers


- Take the seat (H) and turn it upside-down so that the screw holes are facing upwards.
- Take the leg receiver labeled “#1” (L1) and place it into the groove labeled “#1” in the seat with the leg receiver holes pointing upwards.
- Take the leg receiver labeled “#2” (L2) and place it into the groove labeled “#2” in the seat with the leg receiver holes pointing upwards.
- Insert screw (A) and washer (E) into the center screw hole through both leg receivers but do not fully tighten the screw.
- Secure Leg Receiver #1 using 2 flat head screws (B).
- Secure Leg Receiver #2 using 2 flat head screws (B).
- Tighten the center screw.

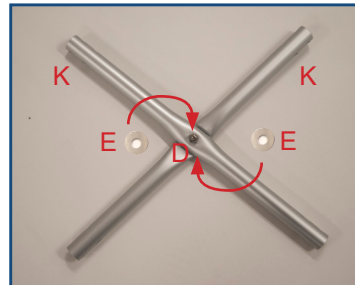


2. Assemble the leg cross-brace

- Take the two leg cross-braces (K) and place one on top of the other with the grooves properly aligned.

- b. Place 1 washer (E) on the top side of the cross-braces. Insert screw (D) through washer (E) and into the center hole and secure with a nut (F).

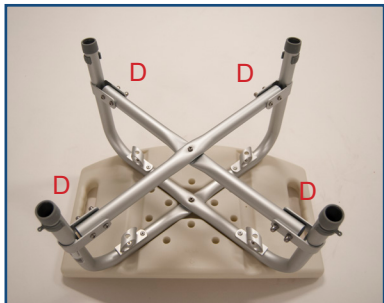


- c. Position the cross-brace assembly in between the four legs and align the cross-brace screw holes with the cross-brace bracket screw holes.
- d. Insert a screw (D) into each of the 8 screw holes and secure with a nut (F).

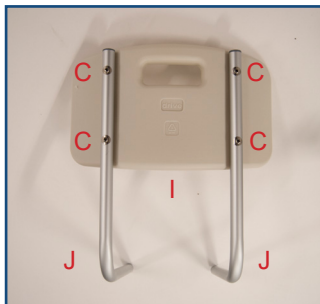

3. Assemble backrest

- a. Take the backrest and the two L-shaped tubes (J).
- b. Place each of the tubes into the grooves on the rear face of the backrest (I).
- c. Align the 2 screw holes in the L-shaped tubes with the holes in the rear face of the backrest.
- d. Place a washer (E) over each of the screw holes and secure the L-shaped tube to the backrest using 2 screws (C) for each tube.

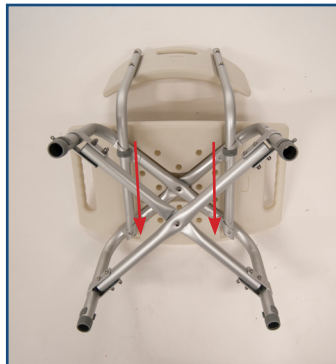

- e. Insert the backrest assembly tubes into the back rest receiver on the underside of the seat.
- f. Secure the backrest assembly to the seat using 2 U-shaped clips (G).

4. Assemble legs

- a. Insert a leg (M) into each of the leg receivers.
- b. Adjust each leg to the same height and align the height-adjustment holes in the legs with the holes in the leg receivers.
- c. Use a U-shaped clip to secure each of the legs to the leg receiver.



Item # 12021KD-1

adjusting leg height

1. Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height. To adjust leg height, remove U-Clip from adjustment hole, set leg to desired height and return U-clip to adjustment hole. Make sure U-Clip clicks into place. Repeat for remaining legs.

NOTE: Make sure all legs are set to the same height.

2. To be sure legs are locked into place, push down or pull up on legs.

additional precautions

1. Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
2. Make certain that the U-Clips full protrude through the same respective holes of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
3. Ensure ALL screws, nuts and/or bolts are tightened.
4. Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
5. All four leg extensions with rubber tips MUST touch the floor simultaneously at all times.
6. DO NOT stand on chair.

NOTE: To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

maintenance

Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, or loose, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit www.drivemedical.com/help for more info.

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2020 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China