drive
WWW.DRIVEMEDICAL.COM

toilet safety frame

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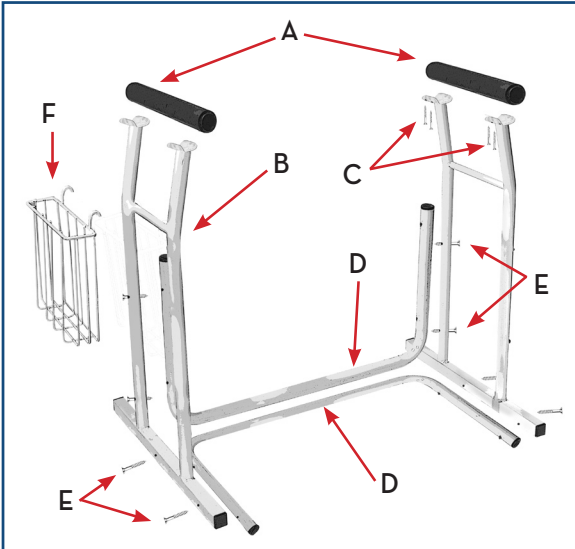
EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

drive
WWW.DRIVEMEDICAL.COM

parts list

- A. Armrest/Handle
- B. Side Frame
- C. Screws
- D. U Shaped Support Frame
- E. Thumb Screws
- F. Basket

assembly instructions

WEIGHT LIMIT 300 LBS.

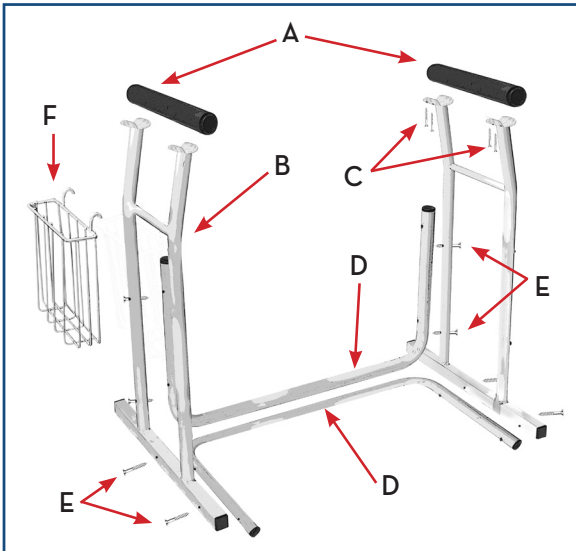
1. Remove contents from carton.
2. Attach armrest/handle (A) to side frame (B) using 2 screws (C). Repeat for second side frame.
3. Stand the side frame (B) on its base. Attach one of the U shaped support frames in the horizontal position (D) to bottom of base using 2 thumb screws (E).
4. Attach the second U shaped support frame (D) to the side frame in vertical position using 2 more thumb screws (E).
5. Repeat steps 3 and 4 for second side frame.
6. Hang magazine basket (F) on desired side of side frame.

Note: Check thumb screws periodically for tightness.
Tighten as necessary.

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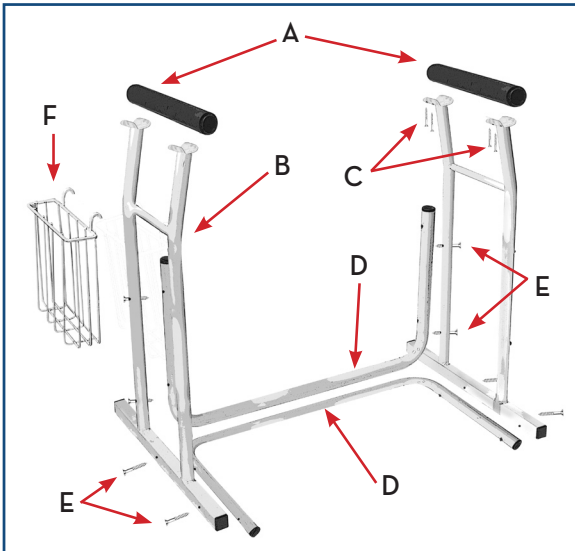
lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

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