MOBILITY DESIGNED

Multi Tub Rail

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Riel Múltiple
Para Bañera

Instructions For Use
Instructions D'utilisation
Instrucciones De Uso

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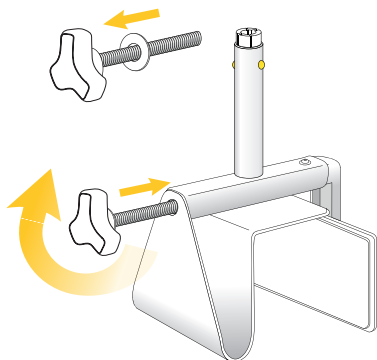
VER.A.10.25


Assembly Instructions

- 1 Clean bathtub thoroughly and wipe dry before applying rail. Contact surfaces must be dry and free of soapy residue to assure secure installation.

- 2 Assemble gripping clamp by pushing the mobile gripping clamp plate inward until the screw grooves are visible. Place the white plastic washer on the adjustment knob screw, near the head of the screw. Insert the screw in the gripping clamp hole and rotate until it has been properly gripped by the threads.

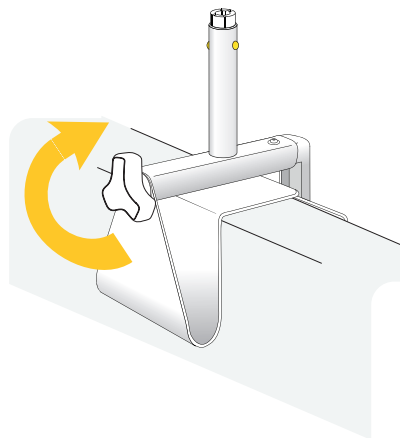
- 3 Lock the gripping clamp onto the tub by placing the gripping clamp on the bath tub wall, at the desired location. The adjustment knob must be positioned outside the tub. Tighten the adjustment knob until the fixed and mobile parts of the clamp become snug. Carefully continue to tighten the clamp until the bath rail is securely attached to the tub wall.

CAUTION: Do not over-tighten as this may damage your tub.

Assembly Instructions

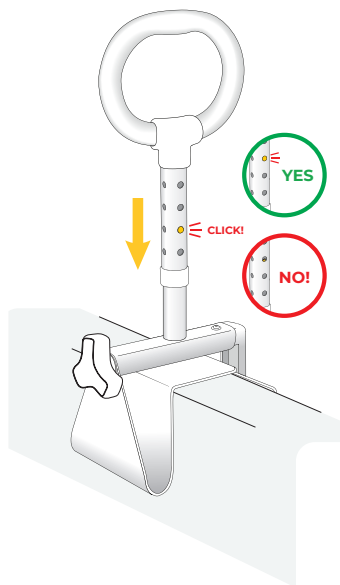- 4** Add the handset extension by placing the multi-orientation height adjustable handgrip onto the metal clamp shaft, as illustrated. Lower the handgrip until the push buttons lock it into place. Adjust the handgrip at the desired height and orientation (parallel or perpendicular to the bath) by pressing the push buttons simultaneously and sliding it on the primary metal shaft. The push buttons will automatically deploy and lock the handgrip into place when a “click” sound is heard.

- 5** Test the mounting/stability of the bath tub rail by pulling on it from side to side and from front to back. If it moves, tighten again in the desired position.

Safety Rail often requires retightening from time to time. Check periodically before use and retighten as needed.

Additional Precautions

WEIGHT CAPACITY: 300 LB / 136 KG

- 1** Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
- 2** Be careful to slowly tighten the adjustment knob to gradually engage the Gripping clamp. Do not over-tighten on glass and/or porcelain tubs as it may crack or damage them.
- 3** Verify that the bath safety rail is properly secured by pulling it from side to side and from front to back. If it moves, tighten again in the desired position. Your safety rail may require tightening from time to time. Check periodically and tighten again when needed.
- 4** Use a mild household cleaner to maintain your product. Avoid the use of harsh solvents, abrasive powders and scouring pads.

NOTE: To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath safety rail dry after each use.

Maintenance and Cleaning

Brush off dust and dirt or wipe with a damp cloth. Clean regularly by hand with a damp cloth and mild detergent only. Do not use aggressive detergents such as chlorine, bleach or abrasive cleaners. M+D by Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, loose, or rusted, and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit www.drivemedical.com/help for more information.

Lifetime Limited Warranty

Your M+D by Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your M+D by Drive device or this warranty, please contact an authorized Drive dealer.

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
This image shows a single sheet of white paper with horizontal blue or grey ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

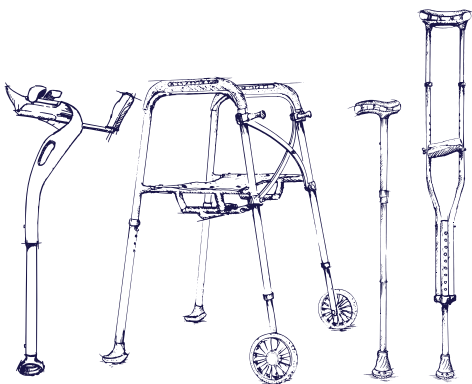[illegible]

© 2025 Medical Depot, Inc. dba Drive DeVilbiss Healthcare.
All rights reserved.

Mobility Designed® is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA
Made in China

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© 2025 Medical Depot, Inc. exerçant son activité sous le nom
Mobility Designed® est une marque déposée de Medical Depot, Inc.
Mobility Designed® es una marca registrada de Medical Depot, Inc.