PreserveTech™ 360° Swivel Bath Chair

Silla Giratoria de Bañera
PreserveTech™ 360°

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Item # RTL12A001-GR

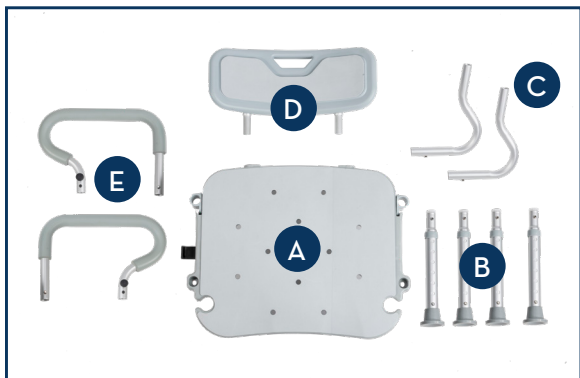
OSMUNDA Medical Technology Service GmbH
Georg-Sigismund-von Oppen-Weg 15,
14476 Postdam, Germany

www.drivemedical.com

drive™

parts/components


- A. Seat
- B. Legs
- C. Backrest Supports
- D. Backrest
- E. Arms


WEIGHT CAPACITY: 350 LB / 158 KG

assembly instructions

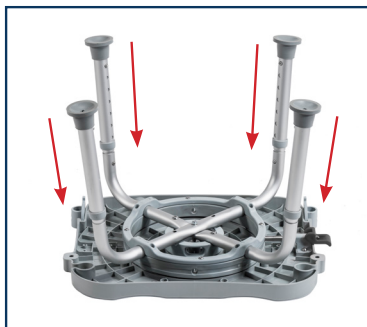1. Turn seat (A) upside down and insert legs (B) into leg receptacles by depressing push pins and sliding legs into the leg receptacles. Push pins will click into place when secure.

2. Attach backrest supports (C) to the underside of the seat (A) by sliding the “L” shaped sections through the receptacles on the bottom of the frame. Push pins will click into place when secure.

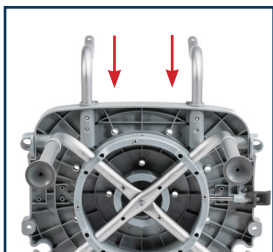3. Attach backrest (D) to backrest supports (C) by depressing push pins on backrest and sliding into place. Push pins will click into place when secure.

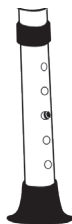
4. Insert arms (E) into the arm receptacles located on the sides of the chair by depressing push pins and sliding into place. Push pins will click into place when secure.

5. To operate, simply pull up on the lever and begin to swivel. The seat will automatically lock at 90° intervals. The swivel shower chair can rotate a full 360° in either direction.

adjusting leg height

- Before use, adjust height of each leg to your specific needs, making sure all legs are at the same height (denoted by the numbers on each leg).
- When desired height is reached, lock legs into place by aligning push pins into proper holes, per illustration. Push pins will click into place when secure.

additional precautions

- Inspect all parts for shipping damage before assembly. If there is shipping damage - DO NOT USE. Contact dealer/carrier for further instructions.
- Make certain that the height adjusting push pins fully protrude through the same respective hole of each leg extension. This will ensure that the leg extensions are securely locked in position and an even height is achieved.
- Ensure ALL screws, nuts and/or bolts are tightened.
- Check the rubber tips on the leg extensions for rips, wear or if they are missing. Immediately replace any or all if any of these imperfections exist.
- All four leg extensions with rubber tips MUST always touch the floor simultaneously.
- When used in a tub, make certain the rubber tips are not touching the sides of the tub wall.
- DO NOT stand on the chair.

maintenance

Drive recommends periodic visual inspection of this product by the user, to ensure that all parts and hardware are secure, that components are in good working order and not worn, torn, frayed, loose, or rusted and that there are no obstructions or impediments to normal, safe operation. If any part or mechanism appears to be unsafe or compromised, do not use the product. If you experience problems, please contact the provider that sold you the product or visit www.drivemedical.com/help for more info.

NOTE: To maintain the beauty of the original finish, clean with mild soap and rinse with warm water. Wipe bath seat dry after each use.

Drive PreserveTech™ products are manufactured with an antimicrobial* agent that helps to minimize the growth of stain and odor-causing bacteria and mold on treated surfaces.

*Antimicrobial properties are built in to protect the products. These products do not protect users or others against bacteria, viruses, germs or other disease organisms

lifetime limited warranty

Your Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.

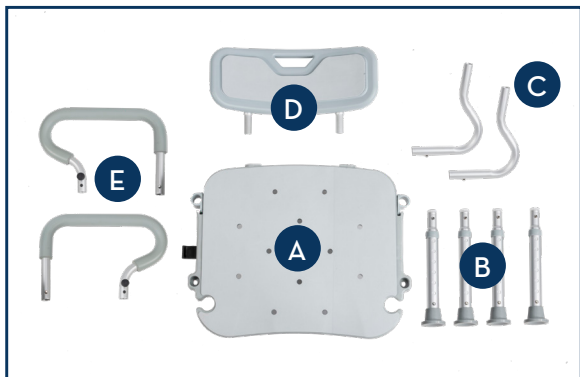
© 2020 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

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© 2020 Medical Depot, Inc. Tous droits réservés.
Drive est une marque de commerce de Medical Depot, Inc.