VITATURN K / VITATURN L

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Table of contents

Preface	7
Declaration of conformity	7
Service life	7
Disposal	7
Identification Plate	7
Reuse.....	7
Warranty	7
Parts of the product	8
Indication.....	8
Scope of delivery	8
Safety instructions.....	8
Use	8
Cleaning Maintenance.....	9
Technical data.....	9

Preface

Thank you for choosing a product from Drive DeVilbiss.

You will not be disappointed by the design, functionality and quality of this product.

Please read these operating instructions carefully prior to initial use. They contain important safety information and valuable tips for the proper use and care. If you have questions or need additional information, contact your Drive DeVilbiss specialist dealer who supplied your product.

These operating instructions contain all of the important information for adjustment and operation. The most current operating instructions are available as a PDF online or can be requested from customer service.

Enlarged formats for visually impaired persons can be displayed. Blind patients must be instructed by their attendant. Repairs as well as some adjustments require special technical training and must therefore be made by the medical supplies dealer.

Declaration of conformity

For the product described, DRIVE MEDICAL GMBH & Co. KG declares compliance according to the medical device regulation (EU) 2017/745.

Service life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed.

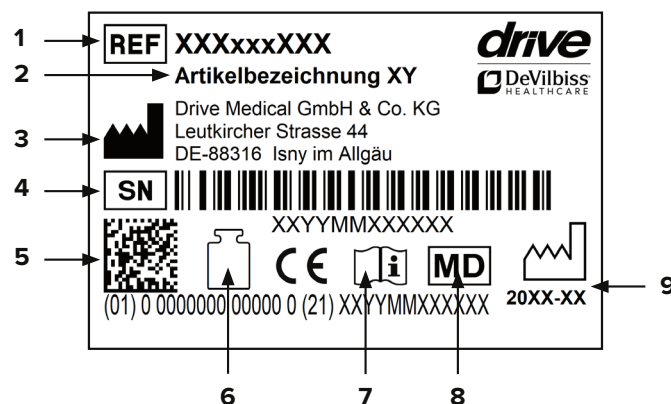
This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however. Determination of the service life by our company does not represent any additional guarantee.

Disposal

If you no longer need the product, please contact your specialist dealer or bring the product to your local disposal center.

Identification plate

1 Article number | 2 Article name | 3 Manufacturer | 4 Serial number | 5 UDI code | 6 Max. load | 7 Follow the instructions for use | 8 Medical device | 9 Production date (year-month)

Reuse

This product can be reused.

Please note that all technical documents needed for safe handling must also be given to the new user.

The product must be checked by the specialist dealer, cleaned, and restored to optimum condition.

Always follow the manufacturer's instructions during preparation; these are made available on request.

Accessories are available from your specialist dealer or at www.drivedevilbiss.de

Warranty

The warranty period for the product supplied by us is 24 months from the date of purchase. Defective parts will be replaced free of charge if it can be shown that there is a defect in materials or manufacturing.

Parts subject to wear are generally not covered during the normal guarantee period unless the parts need repair or replacement which became necessary as a clear, direct consequence of a manufacturing or material error.

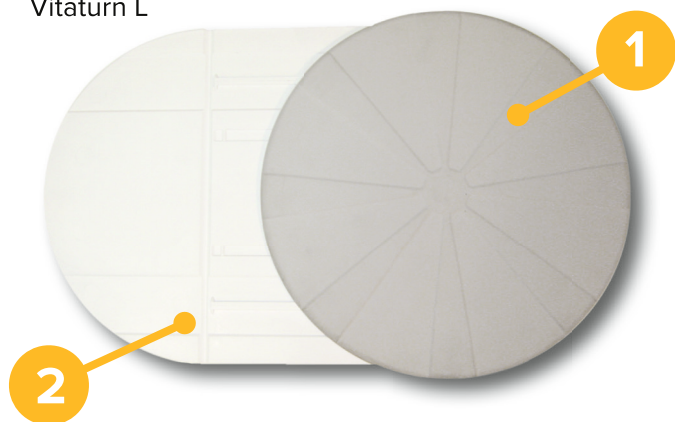
We reserve the right of error and modifications on the basis of technical improvements and changes in design.

Parts of the product

Vitatur K

Vitatur L

1. Swiveling disc
2. Base plate

Indication

INDICATION:

Significant to complete impairment of the ability to change body position and walk, generally in the case of neuromuscular damage or neurological diseases

- In the case of an intact and sufficient ability to stand for short periods of time
- To facilitate changing position, where applicable, with support from an assistant/caregiver, for example, when transferring onto a bathtub lifter or a shower stool

CONTRAINDICATION:

Do not use in cases of insufficient ability to stand or sit for short periods of time.

Scope of delivery

1 x	Swivel aid
1 x	Instructions for use

Safety instructions

- Read these instructions for use carefully.
- All components should be checked for damage and a secure fit prior to use. Do not use product if there are any defects!
- Caution – risk of slipping/falling!
- Take note of the permitted maximum weight of the user. This information can be found in the technical data in these instructions for use as well as on the identification plate.

Use

The swiveling disc is placed on the bath lifter or on the floor. The user sits on or places his/her feet on the disc and is brought into a seated (or standing) position, if necessary with support from an assistant. This is done utilizing the patient's remaining strength in the torso and upper body. In this seated or standing position, a rotating movement on the disc and thus a change in position is now possible.

Cleaning | Maintenance

- For cleaning, the swiveling disc can be easily removed from the base plate; this disc is only clipped into place and is pressed back into place after cleaning (**photo 4-6**).

- Clean regularly by hand with a moist cloth or place in the dishwasher!
- Do not use any aggressive cleaning agents!
- Do not use abrasive cleaners!
- Do not use any high-dose chlorine and vinegar-based cleaners!
- To disinfect, a spray disinfectant can be used such as Descosept AF or Bacillol AF.
- Check the screws regularly and tighten, if necessary.
- The product should not be exposed to temperatures above 60°C for long periods of time (for example, use in a sauna, etc.).
- In the event of defects, contact your specialist dealer from whom you purchased this product.

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Drive Medical GmbH & Co. KG